




PathSuite Software 2.0

Quick Start Guide

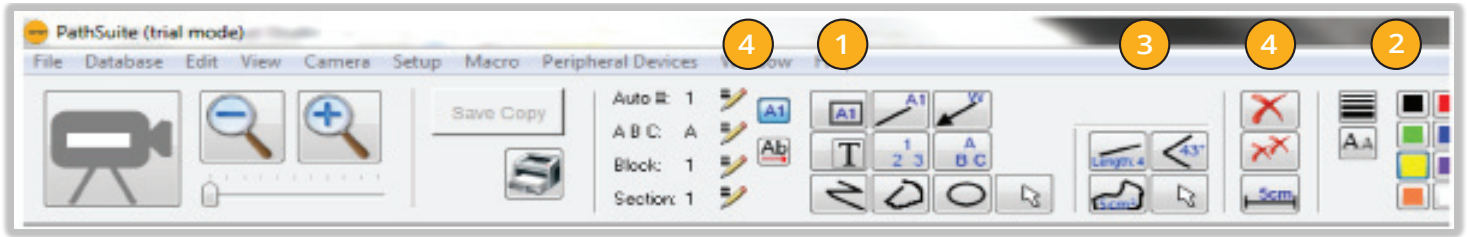

1 Annotation Tools:

Click a tool; then use the mouse to draw that type of annotation.



Block: drag out a rectangles. Labeled with specimen and sequential block # (e.g., A1, A2, A3)



Section: Drag out a line, labeled with specimen and sequential block # (e.g., A1, A2, A3)



Arrow: drag out an arrow; you will be prompted for a label



Text: Drag out rectangle into which you can enter text



Auto-number: Click on a spot to place number there, starting with 1; then 2, 3 etc...



Auto-Alpha: Click on a spot to place a letter there, starting with A; then B, C etc....



Polyline: Click on multiple spots to draw multi-segmented line; double click to finish drawing it



Freehand curve: Drag out a curve; release mouse to finish drawing it



Ellipse: Drag out an Ellipse



Selector tool: While this tool is active, you can click on an object to select it for editing, or drag a rectangle to select a few objects. Then delete via Delete tool, or reposition/re-size via the mouse.

Works only for annotation objects; to select a measurement object, See Measurement tools.

2 Line Width and Color:



Change line thickness and font size for future objects



Change color of future objects

3 Measurement Tools:



Length: Click in two different spots to draw a line between them, labeled with the distance



Angle: Click in 3 spots to form an angle; it will be labeled with angle's labeled



Area: Click, drag out a shape, and click again to complete the shape; it will be labeled with it's area and Perimeter



Selector tool: While this tool is active, you can click on a measurement to select it for editing. Move the end points and the measurement updates automatically.

Works only for measurements.

4 Annotation Options:

Displays the next number to be used with block, section, auto-number and auto-alpha tools. Click the pencils to change them.



Toggles when using blocks, sections and arrows include labels



Change orientation of text in all future objects



Delete selected object



Delete all objects



Toggle calibration mark on and off


Additional Annotation Features

Text Stamps


1. Click the program settings button on the lower-right corner of the screen



2. Click the "Annotations and Measurements" tab of the resulting dialog box.

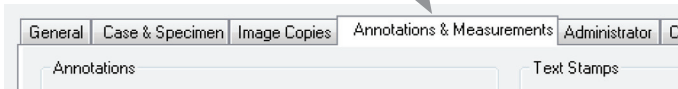


3. Click on "Use Text Stamps"

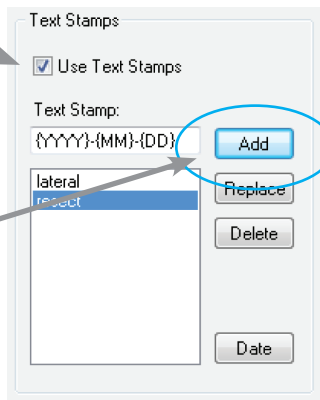


4. Enter a word or phrase into the box and click "Add".

5. Repeat for each phrase you want available as a text stamp.

6. To use an automatically formatted date, click "Date" and use the codes shown in the "Date Codes" dialog.

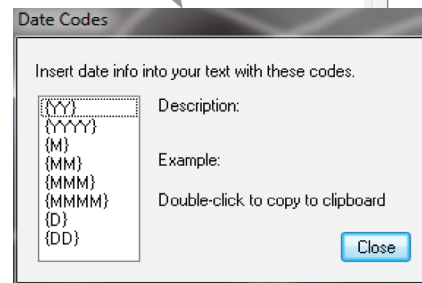

7. When you edit an image, the text-stamp tool will appear next to the other annotation tools. Choose a stamp from the dropdown list, and click in each spot where you want to add that text.

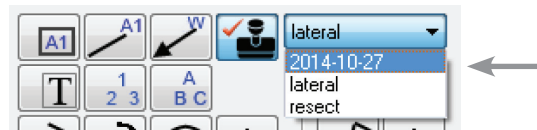

Automatic Corner Text

To include text automatically in every image that is captured, enter it into one of the corner-text boxes. To include dates or case info, click "Case" and "Date".



6540 Burroughs Avenue • Sterling Heights • MI • 48314-2133

phone: 586.731.6000 • fax: 586.731.6469

website: www.spotimaging.com PN3134 Rev.1